




1. To open, remove the tip of the cartridge (at designated groove) with nippers or knife.
2. Ensure a clean straight cut along groove.
3. Slide the red oval finger ring over top of cartridge and seat at cartridge base.
4. Use the red plunger to squeeze out a small amount of material to ensure both sides are dispensing evenly.
5. Attach mixing tip by pressing and twisting clockwise.
6. For additional applications, remove old tip, re-equalize the cartridge, and attach another mix tip.

CARTRIDGE PREPARATION

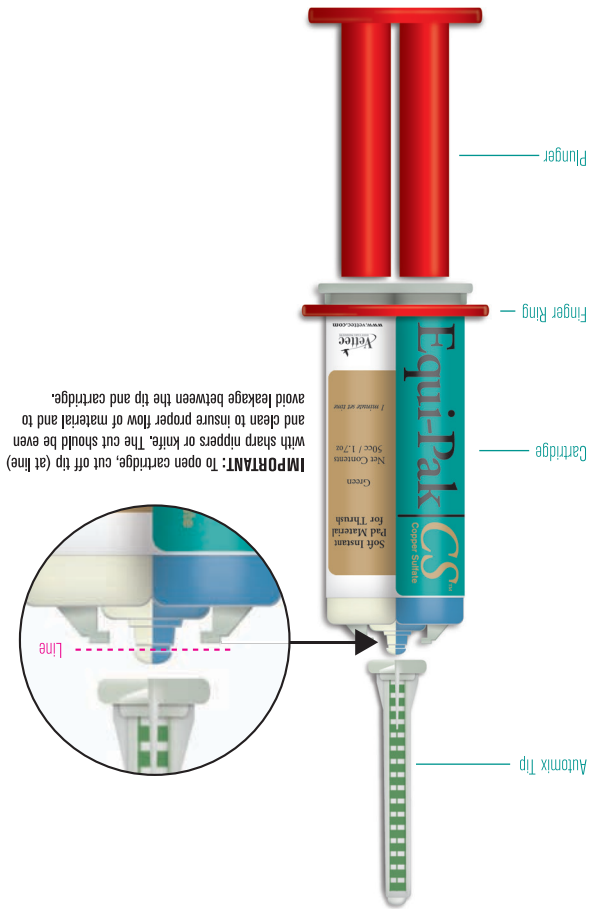

IMPORTANT: To open cartridge, cut off tip (at line) with sharp nippers or knife. The cut should be even and clean to insure proper flow of material and to avoid leakage between the tip and cartridge.

CARTRIDGE PARTS




Since the early 1990's Vettec has introduced a number of innovative hoof care products that have become staples in the hands of hoof care professionals. Vettec's fast setting hoof adhesive products allow the professional to successfully complete hoof repairs, glue on shoes, and create "instant shoes" and foal extensions. Vettec also manufactures a number of "instant pad materials" that promote heel and sole growth, protect and support the equine foot, and comfort a sore-footed horse. If your horse is experiencing hoof related problems contact your hoof care professional or call us at 1.800.483.8832 or visit www.vettec.com.



DP481024-01

USA: 600 East Hueneme Road Oxnard, CA 93033 USA 1.800.483.8832
 Europe: Zonnebaan 14, NL-3542 EC Utrecht, The Netherlands +(31) 30 241 1823

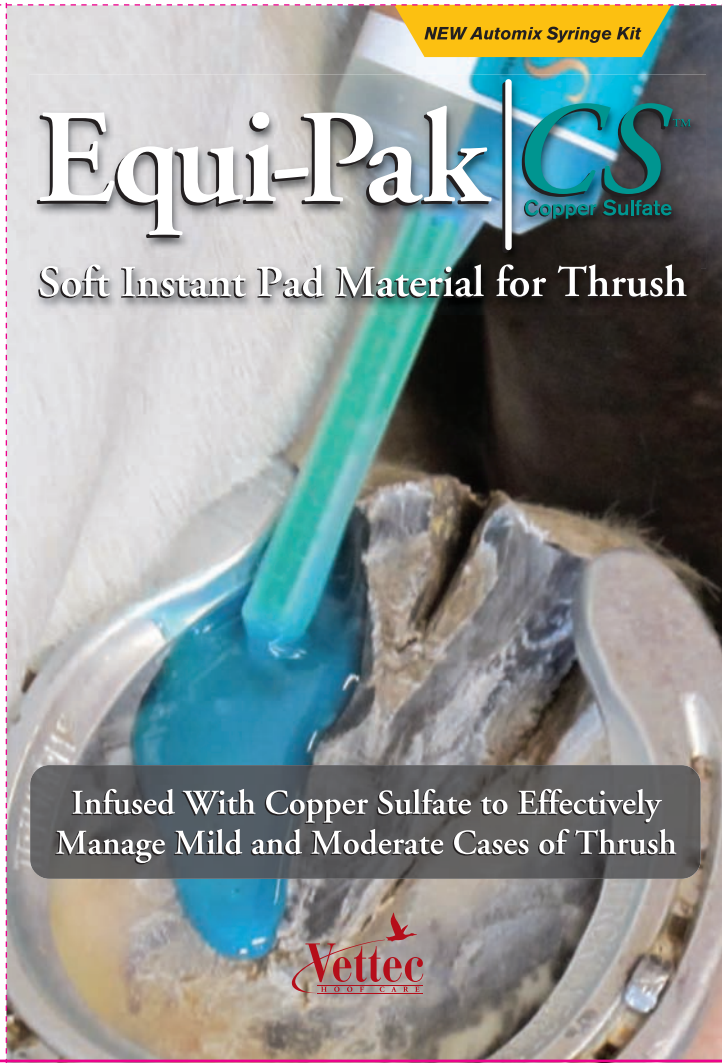

Equi-Pak | CS™

Copper Sulfate

Soft Instant Pad Material for Thrush

Infused With Copper Sulfate to Effectively Manage Mild and Moderate Cases of Thrush



Equi-Pak | CSTM

Copper Sulfate

Soft Instant Pad Material for Thrush

PLEASE NOTE

- Vettec products are best used between 65°- 85° F (18°- 29° C).
- Areas of bleeding, abscesses or severe thrush should not be covered. Once healed, commence using products.
- Use of disposable gloves is recommended.
- Do not burn or "Hot Shoe" this product.
- Use only Vettec mix tips.
- This procedure should be performed on a clean, flat work area.

Equi-Pak CSTM is a fast setting (45 seconds) liquid urethane hoof packing material infused with copper sulfate for the treatment and prevention of thrush. Equi-Pak CS protects both the frog and sole and eliminates the need to pick out the feet or apply daily medication.

HOOF PREPARATION

1. Remove all loose sole and dirt with a wire brush.
2. Clean frog and commissures. Remove any dirt or debris where the shoe meets the sole.
3. Have all materials ready before proceeding to the next step.
4. If possible dry sole with heat gun or hair dryer immediately before applying Equi-Pak|CS.



EQUI-PAK|CS APPLICATION

1. Dry the sole and frog thoroughly with a heat gun or hair dryer (if possible).
2. Nail the shoe into position. If the sole is flat, seating out the shoe is advised.
3. With the frog turned upward, apply Equi-Pak|CS underneath the web of the shoe and fill in the sole until the desired level is achieved. This is usually to the top of the shoe.
4. Hold the hoof until the Equi-Pak|CS sets, approximately 45-60 seconds, then release.
5. Trim any excess material.
6. Allow 10-15 minutes before riding.

