



Equi-Pak Soft™ (REF#s 46118) is a fast setting, 30 second, liquid urethane hoof packing material that provides shock absorbent, cushioned support that retains its shape and flexibility indefinitely.

Equi-Pak Soft should be used with a pad on working horses and adheres to the sole sealing out moisture and debris, protecting both the frog and sole. **Equi-Pak Soft** can be filled to ground level for lameness cases. Equi-Pak Soft will leave the sole in excellent condition, eliminating cheesy sole.

PLEASE NOTE

- Vettec products are best used between 65° and 85° F (18° to 29° C).
- Areas of wetness, bleeding, or infection should not be covered. Once healed, commence using products.
- Do not burn or “Hot Shoe” this product.
- Use only Vettec mixing tips.
- Keep the Dispensing gun clean.

CARTRIDGE PREPARATION

1. To open the cartridge, remove the tip at the line with nippers. Insert cartridge into dispensing gun, label side up.
2. Before attaching the mixing tip, equalize the cartridge by squeezing out a small amount to ensure that both sides are dispensing evenly.
3. Attach mixing tip by pressing and twisting clockwise.
4. When finished, don't release the pressure with the lever on the back of the dispenser. Remove the mixing tip immediately, discard, and replace with the cap. Push the pressure release lever. Caps are available from your dealer.
5. For additional applications, re-equalize the cartridge, and attach a new tip.

HOOF PREPARATION

1. Trim and conservatively pare the sole. Clean well with a wire brush.
2. Clean up frog and commissures.
2. **DRY HOOF THOROUGHLY** using a heat gun, even if you don't think it needs it. Don't overheat.

PROCEDURE

With Pads (for horses in work)

- 1a. **If open at heels:** Place mixing tip under the pad at the heel and fill to back of the pad.
- 1b. **If there is no access:** Pre-drill a 5/16 inch hole in the pad near the apex of the frog and nail into position under the shoe. Place mixing tip against the hole and dispense until filled to the back of the pad.
Cover the hole with a small piece of duct tape.
2. Once set, in approximately 30 seconds, release the hoof.
3. Replace mixing tip and repeat process as necessary.

Without Pads (for therapeutic lameness cases)

1. After preparing the sole per directions listed above, wire brush the sole and dry thoroughly with a heat gun or hair dryer.
2. Cut a piece of **Equi-Mesh** (REF #46030) to the outline of the shoe and place between the hoof and shoe, putting the curved side toward the sole.
3. Nail the shoe into position. **If the sole is flat, seating out the shoe is advised.**
4. With the frog turned upward, apply Equi-Pak Soft underneath the web of the shoe and fill in the sole until the desired level is achieved. This is usually to the top of the shoe.
NOTE: For added support, load to ground level using a **Foamboard** (REF # 46015).
5. Hold the hoof until the Equi-Pak sets, approximately 30 seconds, then release.

Limited Remedy: Follow instructions carefully. If your application is not covered by these instructions, call 800/483-8832 before using this product. Buyer agrees that if Vettec products are proved defective, the user's exclusive remedy will be, at manufacturer's election, to replace the defective Vettec product or to refund the purchase price for the defective quantity. Manufacturer is not liable for direct, indirect, incidental or consequential damages regardless of legal theory asserted, including assertion of negligence and strict liability.